

**ProAir®
digihalter®**
(albuterol sulfate 117 mcg)
Inhalation Powder

**AirDuo®
digihalter®**
(fluticasone propionate
113 mcg and salmeterol
14 mcg) Inhalation Powder

**ArmonAir®
digihalter®**
(fluticasone propionate
113 mcg) Inhalation Powder

Mobile App Instructions

Your Inhaler and Mobile App

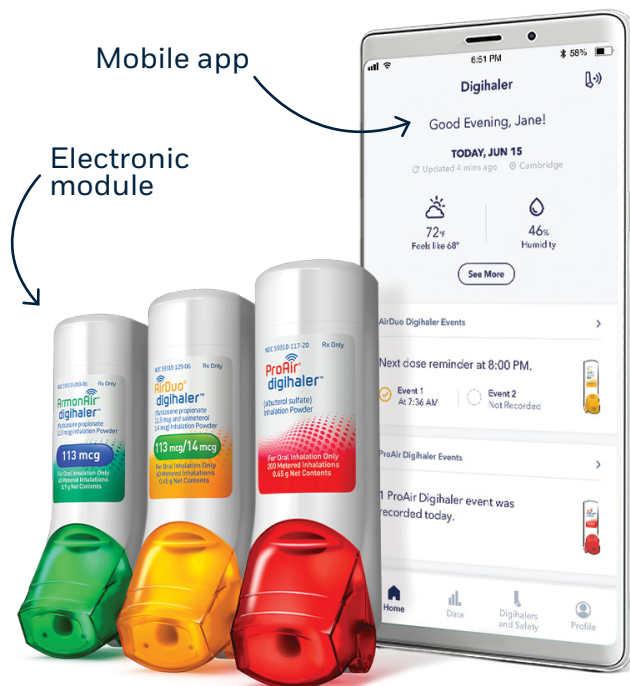


Figure A

- The inhalers in the Digihaler family have sensors that are built into the **electronic module** on top of the inhaler (**see Figure A**)
The sensors in the module
 - Track your inhaler events (recorded in the app as Good Inhalation, Fair Inhalation, Low or No Inhalation recorded within 1 minute of cap open, Exhalation, and Possible Air Vent Block)
 - Measure inhalation airflow, which is the amount of effort for each inhalation. It is recorded as liters per minute [L/min], and tells you how you are inhaling
- There is also *Bluetooth®* wireless technology built into the module so that the information from your inhaler can be sent to the **mobile app** (**see Figure A**)
 - Your inhaler events will be captured in the app any time your inhaler is connected. The electronic module stores your inhaler events as they happen
 - Bring your inhaler(s) within range of your smartphone regularly, with Bluetooth on and the app open, to update your app with the latest inhaler information
- You don't need the app or need to be near your smartphone in order for your inhaler to deliver medicine

APPROVED USES FOR PROAIR DIGIHALER

PROAIR® DIGIHALER® (albuterol sulfate) Inhalation Powder is a prescription medicine used in people 4 years of age and older to:

- treat or prevent bronchospasm in people who have reversible obstructive airway disease
- prevent exercise-induced bronchospasm

APPROVED USE FOR AIRDUO DIGIHALER AND ARMONAIR DIGIHALER

- **AIRDUO® DIGIHALER®** (fluticasone propionate and salmeterol) inhalation powder is a prescription medicine used to control symptoms of asthma and to prevent symptoms such as wheezing in people 12 years of age and older.
- **ARMONAIR® DIGIHALER®** (fluticasone propionate) inhalation powder is a prescription medicine for the long-term treatment of asthma in patients 12 years and older.
- **AIRDUO DIGIHALER AND ARMONAIR DIGIHALER** are not used to relieve sudden breathing problems from asthma and won't replace a rescue inhaler.

Please see Important Safety Information for all products throughout.

Click here for full Prescribing Information: [ProAir Digihaler](#), [AirDuo Digihaler](#), and [ArmonAir Digihaler](#).

Downloading Your Digihalter App

- In order to use the app, you will need to download and install it on your smartphone. An Internet connection is required to download the app
- Search for “**Digihalter**” in the App Store® or Google Play™
- Download the free app
- You may need to update the Digihalter app when newer versions are available



Sign Up and Log In

Figure B

Once you have installed the app, you will need to:

Create an account or login. You will need an Internet connection to do this.

- After downloading the app, tap “Get Started” on the Welcome Screen
- Enter your date of birth and accept the Terms of Use and Privacy Notice to continue
 - You must be 18 years or older to create an account
- You will then be asked for permission to use “amazonincognito.com” to redirect you to Teva’s authentication server, for authentication
- If you already have an account, you can log in or tap “Need to Sign Up?” to create a new account **(see Figure B)**
- Lastly, you need to select the patient who will be using the app
 - You can select yourself or add a “Teenage Profile” for your dependent
 - To add a “Teenage Profile,” you will need to specify the first and last names of the teenager and their date of birth

IMPORTANT SAFETY INFORMATION FOR PROAIR DIGIHALER

- **Do not use PROAIR DIGIHALER (albuterol sulfate) Inhalation Powder if you** are allergic to albuterol sulfate, lactose, milk proteins, or any of the ingredients in **PROAIR DIGIHALER**. Ask your healthcare provider if you have any questions or are not sure

IMPORTANT SAFETY INFORMATION FOR AIRDUO DIGIHALER AND ARMONAIR DIGIHALER

- **AIRDUO DIGIHALER** contains salmeterol. Long-acting beta₂-agonist (LABA) medicines such as salmeterol when used alone increase the risk of hospitalizations and death from asthma problems. **AIRDUO DIGIHALER** contains an inhaled corticosteroid (ICS) and a LABA. When an ICS and a LABA are used together, there is not a significant increased risk in hospitalizations and death from asthma problems.
- **Do not use AIRDUO DIGIHALER or ARMONAIR DIGIHALER to treat sudden breathing problems from asthma.** Always have a rescue inhaler with you to treat sudden symptoms.

Getting Started With Your Digihalter App

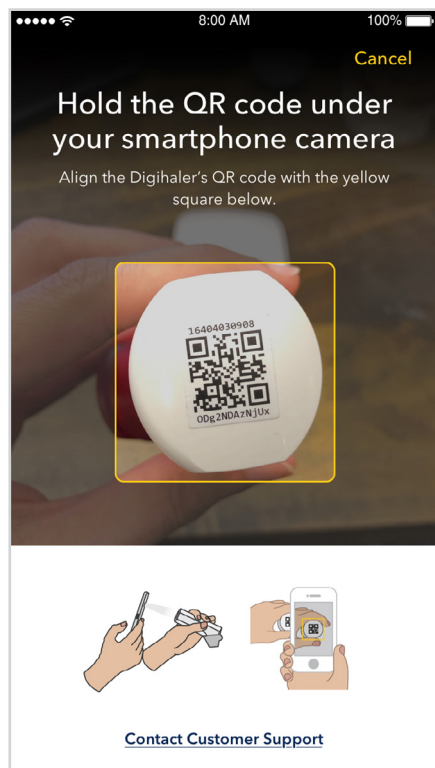


Figure C

Pair your inhaler with the app

- You can scan your inhaler by tapping on “Set Up a Digihalter.” Align the QR code on top of your inhaler behind your smartphone so you can see the QR code in the yellow square on your screen (see Figure C)
 - The first time you attempt to scan an inhaler, you will receive a message asking for permission to use the camera

Name your inhaler

- You can choose one of the names from the list or create your own name
- The app will then show the confirmation screen, which will alert you that your inhaler is registered and ready to use
 - **For AirDuo Digihalter and ArmonAir Digihalter:** You will also be informed about the default reminder times (8AM and 8PM). You can change these reminder times in Notification Settings
- You can add other inhalers at the end of the onboarding walkthrough, which takes place the first time you use your app (see page 4)

IMPORTANT SAFETY INFORMATION FOR PROAIR DIGIHALER

- **Before using PROAIR DIGIHALER, tell your healthcare provider** about all of your medical conditions, including if you:
 - have heart problems
 - have high blood pressure (hypertension)
 - have convulsions (seizures)
 - have thyroid problems
 - have diabetes
 - have low potassium levels in your blood
 - are pregnant or planning to become pregnant
 - are breastfeeding or planning to breastfeed

IMPORTANT SAFETY INFORMATION FOR AIRDUO DIGIHALER AND ARMONAIR DIGIHALER

- **Do not** use AIRDUO DIGIHALER or ARMONAIR DIGIHALER if you have a severe allergy to milk proteins or if you are allergic to any of the ingredients in the product. Ask your healthcare provider if you are not sure.

The Digihaler App Home Screen

On the Home Screen, you will see the following (see Figure D):

Notifications: All notifications regarding your inhaler will be displayed on the top of this screen.

Greeting and Date: A greeting with your name will be displayed along with the date. It will also show your current location (if permission was given to do so) and last updated time.

Medication-specific Information for AirDuo Digihaler or ArmonAir Digihaler: This section will display the number and time of scheduled and recorded AirDuo Digihaler or ArmonAir Digihaler events for that day, if at least 1 inhaler was scanned.

Daily Self-Assessment: This section will allow you to select the face that best represents how you are feeling that day.

Data navigation icon: Tapping on this icon will display all your historical data.

Digihalers Connectivity icon: Tapping on this icon will present a list of all your inhalers and their connectivity status.

Environment and Weather information: The Home Screen will present the default choices – Temperature, Air Quality, and Total Pollen Count – unless you specify which weather elements you are most interested in during onboarding.

Medication-specific Information for ProAir Digihaler: This section will display the number of ProAir Digihaler events for that day, if at least 1 inhaler was scanned.

Digihalers and Safety navigation icon: Tapping on this icon will show all paired inhalers with their given names, connectivity status, strength, and support materials. You can also add other inhalers in this section.

Profile navigation icon: Tapping on this icon will present a menu with options to add a Healthcare Program, view or change notification settings, and reach customer support.

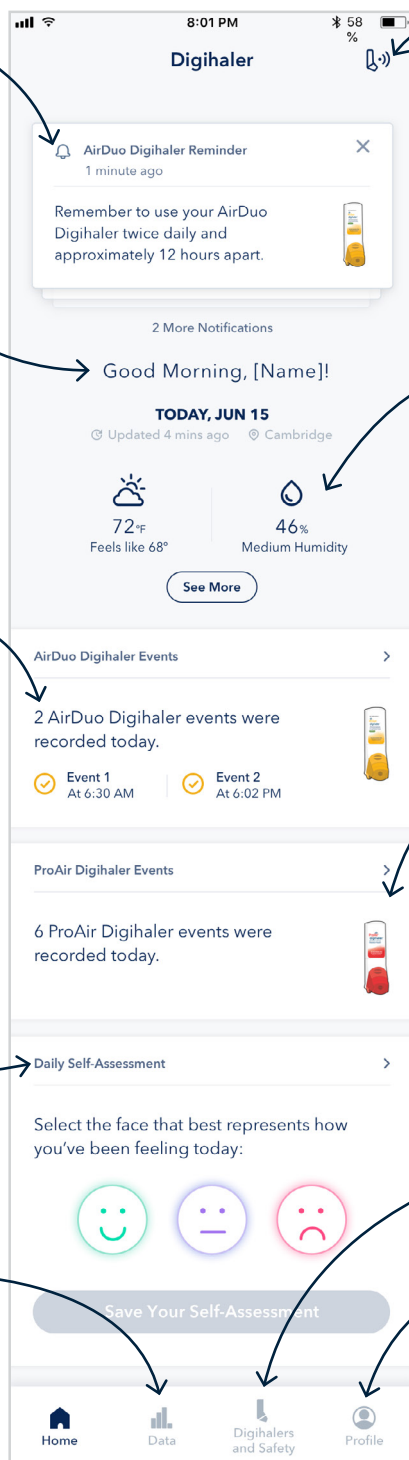


Figure D

Digihaler Support Menu


In the Digihalers and Safety Screen, each of your paired inhalers will have a link to a Support page. The Support page will include the following (see Figure E):

Instructions For Use: See information on how to use your inhaler.

Frequently Asked Questions: View answers to frequently asked questions about your inhaler.

Approved Uses and Important Safety Information: See the Approved Uses and Important Safety Information for your inhaler.

FCC Regulatory Information: View regulatory information for your inhaler.

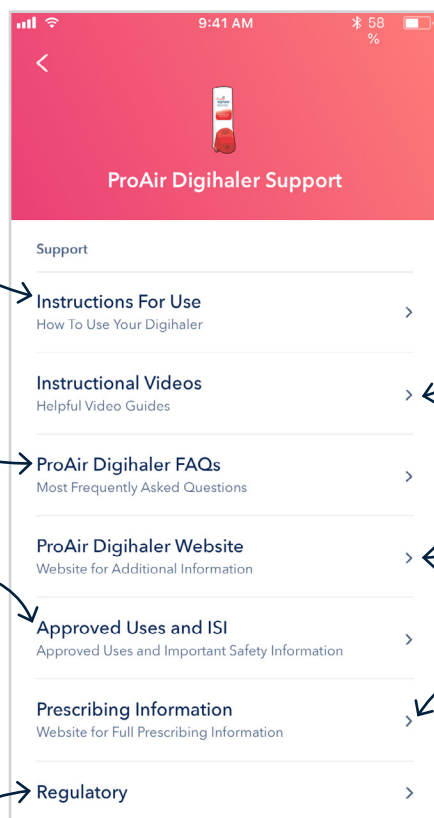

Instructional Videos: Takes you to a selection of videos for your inhaler.

Website: Tapping this option will take you to the website.

Prescribing Information: See Prescribing Information for your inhaler.

Figure E

Illustrated here with ProAir Digihaler but also applies for AirDuo Digihaler and ArmonAir Digihaler.

IMPORTANT SAFETY INFORMATION FOR PROAIR DIGIHALER

- **Tell your healthcare provider about all the medicines you take, especially:**
 - other inhaled medicines or asthma medicines
 - beta blocker medicines
 - diuretics
 - digoxin
 - monoamine oxidase inhibitors
 - tricyclic antidepressants

IMPORTANT SAFETY INFORMATION FOR AIRDUO DIGIHALER AND ARMONAIR DIGIHALER

- **Do not** use AIRDUO DIGIHALER or ARMONAIR DIGIHALER more often than prescribed.
- Do not take AIRDUO DIGIHALER with other medicines that contain a LABA for any reason.
- **Tell your healthcare provider about all the medicines you take and about all of your health conditions.**

Feedback and Notifications

There are several notifications you may see, based on your inhaler events. They will appear as pop-ups on your screen if your app is open. Some will appear as background notifications if your app is closed.

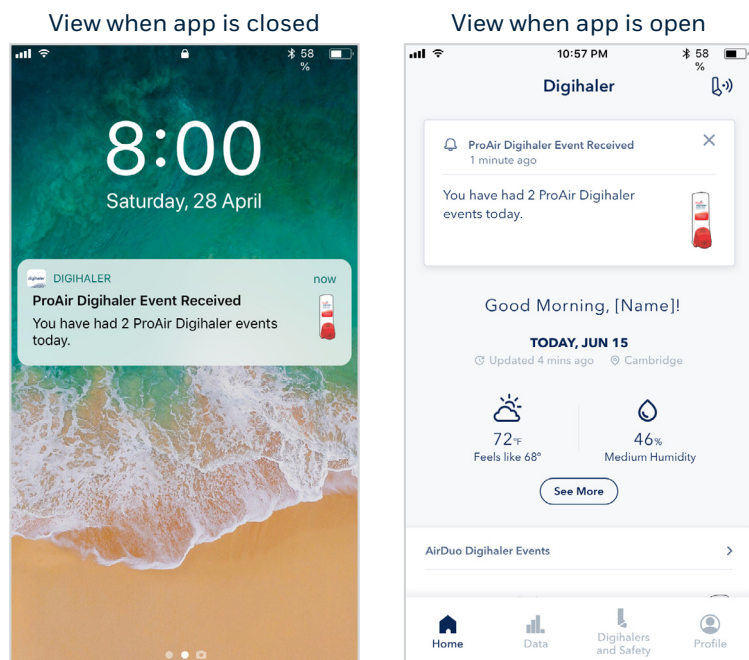


Figure F

If your inhaler is within range of your smartphone within 15 minutes of an inhaler event, you will get a notification confirming that the event has been received by the app **(see Figure F)**.

- You can manage notifications within Notification Settings in the app

Notifications must be enabled in order to receive alerts and reminders when the app is closed.

Illustrated here with ProAir Digihalter but also applies for AirDuo Digihalter and ArmonAir Digihalter.

IMPORTANT SAFETY INFORMATION FOR PROAIR DIGIHALER

- Do not** increase your dose or take extra doses of **PROAIR DIGIHALER** without first talking to your healthcare provider

IMPORTANT SAFETY INFORMATION FOR AIRDUO DIGIHALER AND ARMONAIR DIGIHALER

- Do not** stop using **ARMONAIR DIGIHALER**, even if you are feeling better, unless your healthcare provider tells you to. If you miss a dose of **ARMONAIR DIGIHALER**, just skip that dose. Take your next dose at your usual time. Do not take 2 doses at 1 time.
- AIRDUO DIGIHALER** and **ARMONAIR DIGIHALER** can cause serious side effects, including:
 - Fungal infection in your mouth or throat (thrush).** Rinse your mouth with water without swallowing after using **AIRDUO DIGIHALER** or **ARMONAIR DIGIHALER** to help reduce your chance of getting thrush.
 - Weakened immune system and increased chance of getting infections (immunosuppression).** You should avoid exposure to chickenpox and measles, and, if exposed, tell your healthcare provider right away. Worsening of existing tuberculosis fungal, bacterial, viral, or parasitic infections, or herpes infection of the eye (ocular herpes simplex) may occur.
 - Reduced adrenal function.** This can happen when you stop taking an oral corticosteroid (such as prednisone) and start taking a medicine containing an ICS (such as **AIRDUO DIGIHALER** or **ARMONAIR DIGIHALER**). During this transition period, when your body is under stress such as from fever, trauma (such as a car accident), infection, or surgery, adrenal insufficiency can get worse and may cause death. Symptoms of adrenal insufficiency include:
 - feeling tired
 - Weakness
 - low blood pressure
 - lack of energy
 - nausea and vomiting

Please see Important Safety Information for all products throughout.

Click here for full Prescribing Information: [ProAir Digihalter](#), [AirDuo Digihalter](#), and [ArmonAir Digihalter](#).

Feedback and Notifications (continued)



Figure G

Tapping the Data navigation icon at the bottom of the Home Screen will display data from your inhaler events.

- Event data for all registered inhalers will be presented together if they were scanned into the app
- Selecting a specific inhaler will show you a screen with your inhaler events for the past 7 days, past 30 days, week, or month
 - **For AirDuo Digihaler and ArmonAir Digihaler:** You will also see the anticipated number of events based on how often the medication is taken (twice daily)
- Tapping on a specific day, from either the weekly or monthly view, will give you a breakdown of your inhalations for that day. Events will be listed in order from the first to the last, and are presented in different colored dots based on the inhalation category (**see Figure G and below**)
 - **For ProAir Digihaler:** You will see the events categorized by both color and size. The size of the dots will increase or decrease based on the inhalation category
 - **For AirDuo Digihaler and ArmonAir Digihaler:** You will see the current time settings of your reminders

Events are categorized by:

- Good inhalation (More than 45 L/min)
- Fair inhalation (30 to 45 L/min)
- Low (Less than 30 L/min) or No inhalation recorded within 1 minute of cap open
- Exhalation
- Possible Air Vent Block

It is possible that your inhalation notification may not show up immediately due to wireless transmission interruptions or other issues. You may need to open the app to see notifications.

You must inhale your dose within 1 minute of opening the cap. If nothing happens within 1 minute of opening the cap, it will be recorded as Low (less than 30 L/min) or No Inhalation.

Feedback and Notifications (continued)

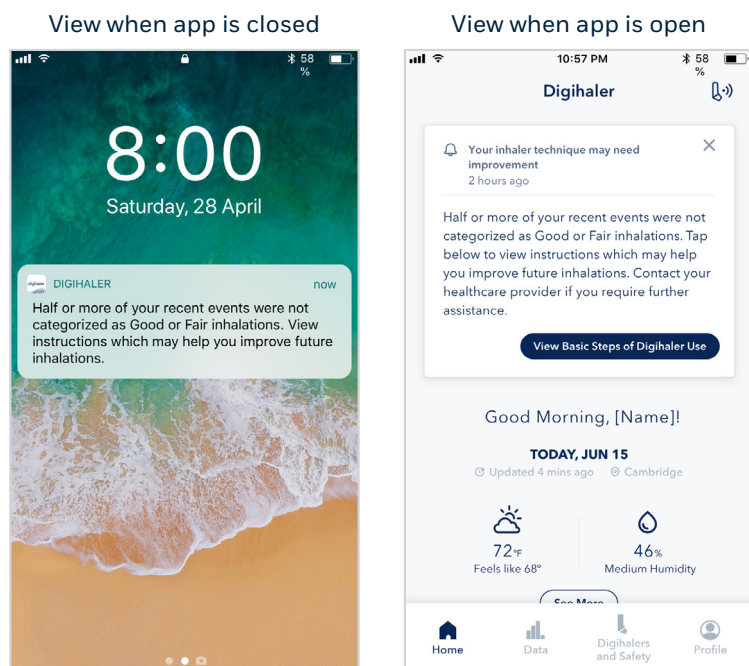


Figure H

You will get a pop-up notification if the app detects that 6 or more of your last 12 inhaler events for all scanned inhalers are not categorized as **(see Figure H)**:

- Good Inhalation (More than 45 L/min)
- Fair Inhalation (30 to 45 L/min)

Tapping on the notification will take you to “Digihaler Basics” to view instructions which may help you improve future inhalations.

You will not get this notification if your last inhalation was categorized as “Good Inhalation” or “Fair Inhalation.”

IMPORTANT SAFETY INFORMATION FOR PROAIR DIGIHALER

- Get medical help right away if **PROAIR DIGIHALER** no longer helps your symptoms, your symptoms get worse or you need to use your inhaler more often

IMPORTANT SAFETY INFORMATION FOR AIRDUO DIGIHALER AND ARMONAIR DIGIHALER

- For **AIRDUO DIGIHALER**, there may be **sudden breathing problems immediately after inhaling your medicine**. If you have sudden breathing problems immediately after inhaling your medicine, stop using **AIRDUO DIGIHALER** and call your healthcare provider right away.
- **Serious allergic reactions**. Stop using **AIRDUO DIGIHALER** or **ARMONAIR DIGIHALER** and call your healthcare provider or get emergency medical care if you get any of the following symptoms of a serious allergic reaction:
 - rash
 - hives
 - swelling of your face, mouth, and tongue
 - breathing problems

**ProAir®
digihalter®**
(albuterol sulfate 117 mcg)
Inhalation Powder

**AirDuo®
digihalter®**
(fluticasone propionate
113 mcg and salmeterol
14 mcg) Inhalation Powder

**ArmonAir®
digihalter®**
(fluticasone propionate
113 mcg) Inhalation Powder

Feedback and Notifications (continued)

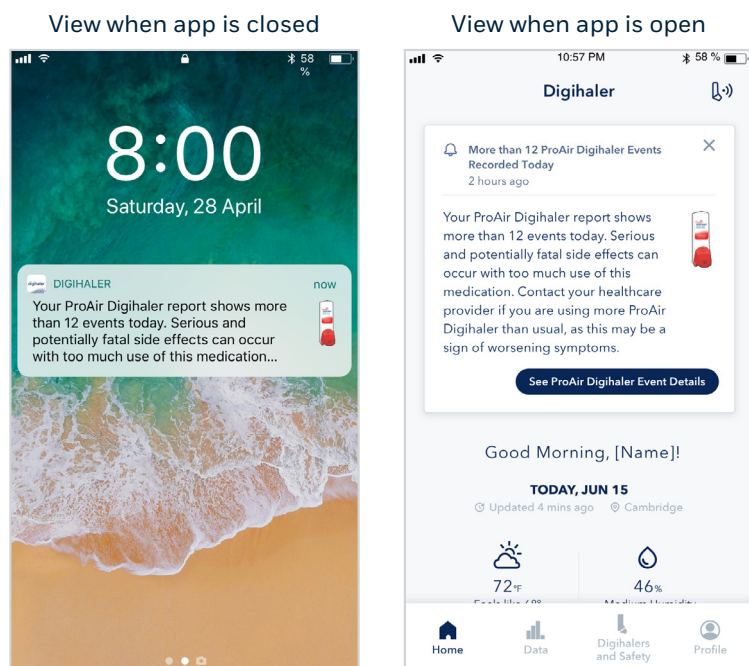


Figure I

You will get a pop-up notification if there have been more than 12 ProAir Digihalter events recorded in one day (**see Figure I**).

- Serious and potentially fatal side effects can occur with too much use of this medication

You will be encouraged to contact your healthcare provider if you are still having symptoms.

You can tap the in-app notification to review the details of your recent events.

IMPORTANT SAFETY INFORMATION FOR PROAIR DIGIHALER

- While you are using **PROAIR DIGIHALER**, **do not** use other inhaled rescue medicines and asthma medicines unless your healthcare provider tells you to do so

IMPORTANT SAFETY INFORMATION FOR AIRDUO DIGIHALER AND ARMONAIR DIGIHALER

- **AIRDUO DIGIHALER** can also cause additional serious side effects, including:
 - **Effects on the heart**
 - increased blood pressure
 - a fast or irregular heartbeat
 - chest pain
 - **Effects on the nervous system**
 - Tremor
 - nervousness
- **AIRDUO DIGIHALER** and **ARMONAIR DIGIHALER** can cause:
 - **Bone thinning or weakness (osteoporosis)**
 - **Slowed growth in children.** A child's growth should be checked often.
 - **Eye problems including glaucoma and cataracts.** You should have regular eye exams while using **AIRDUO DIGIHALER** or **ARMONAIR DIGIHALER**.

Please see Important Safety Information for all products throughout.

Click here for full Prescribing Information: [ProAir Digihalter](#), [AirDuo Digihalter](#), and [ArmonAir Digihalter](#).

**ProAir®
digihalter®**
(albuterol sulfate 117 mcg)
Inhalation Powder

**AirDuo®
digihalter®**
(fluticasone propionate
113 mcg and salmeterol
14 mcg) Inhalation Powder

**ArmonAir®
digihalter®**
(fluticasone propionate
113 mcg) Inhalation Powder

Feedback and Notifications (continued)

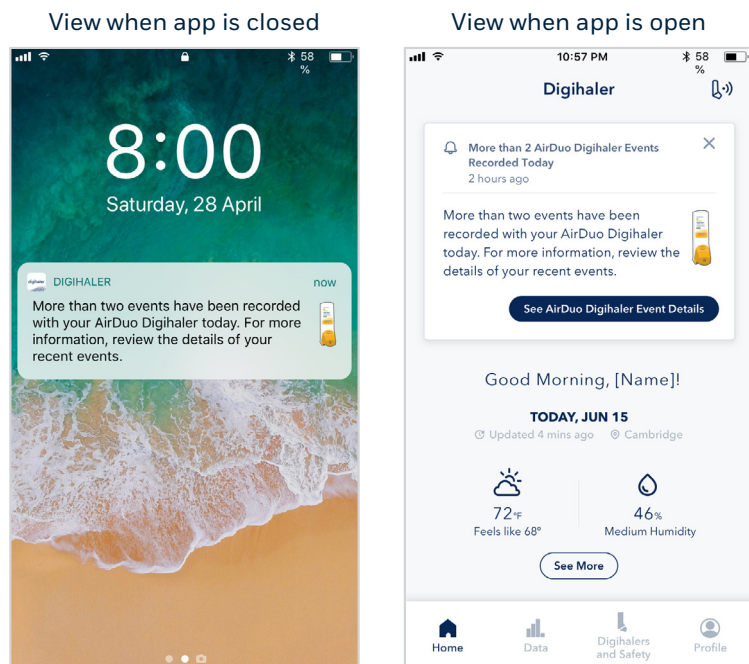


Figure J

You will get a pop-up notification if there have been more than 2 AirDuo Digihalter or ArmonAir Digihalter events recorded in one day (see Figure J).

You can tap the in-app notification to review the details of your recent events.

Illustrated here with AirDuo Digihalter but also applies for ArmonAir Digihalter.

IMPORTANT SAFETY INFORMATION FOR PROAIR DIGIHALER

- **PROAIR DIGIHALER may cause serious side effects, including:**
 - **worsening trouble breathing, coughing and wheezing (paradoxical bronchospasm).** If this happens, stop using **PROAIR DIGIHALER** and call your healthcare provider or get emergency help right away. This is more likely to happen with your first use of a new asthma inhalation medicine
 - **heart problems, including faster heart rate and higher blood pressure**
 - **possible death in people with asthma who use too much PROAIR DIGIHALER**
 - **allergic reactions.** Call your healthcare provider right away if you have the following symptoms of an allergic reaction:
 - itchy skin
 - swelling beneath your skin or in your throat
 - rash
 - worsening trouble breathing
 - **changes in laboratory blood values (sugar, potassium)**

IMPORTANT SAFETY INFORMATION FOR AIRDUO DIGIHALER AND ARMONAIR DIGIHALER

- For **AIRDUO DIGIHALER**, changes may occur in laboratory blood values (sugar, potassium, certain types of white blood cells)
- For **ARMONAIR DIGIHALER** increased wheezing (bronchospasm) may occur. Increased wheezing can happen right away after using **ARMONAIR DIGIHALER**. If this occurs, stop using **ARMONAIR DIGIHALER** and call your healthcare provider. Always have a rescue inhaler with you to treat sudden wheezing.

Please see Important Safety Information for all products throughout.

Click here for full Prescribing Information: [ProAir Digihalter](#), [AirDuo Digihalter](#), and [ArmonAir Digihalter](#).

Feedback and Notifications (continued)

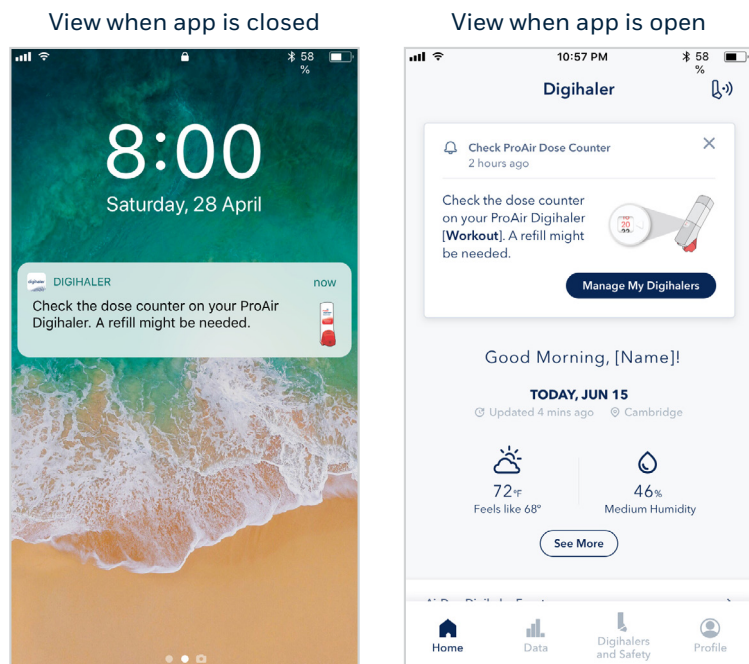


Figure K

You will get a pop-up notification to check the dose counter on your inhaler when a refill might be needed (**see Figure K**).

If you have more than 1 inhaler in this state, the notification will not show an inhaler name.

Illustrated here with ProAir Digihalter but also applies for AirDuo Digihalter and ArmonAir Digihalter.

If your notifications seem inconsistent with your usage, or if you are having difficulties seeing notifications, contact Customer Support at 1-888-603-0788

IMPORTANT SAFETY INFORMATION FOR PROAIR DIGIHALER

- The most common side effects of **PROAIR DIGIHALER** include:
 - back pain
 - body aches and pain
 - upset stomach
 - sinus headache
 - urinary tract infection
 - your heart feels like it is pounding or racing (palpitations)
 - chest pain
 - fast heart rate
 - shakiness
 - nervousness
 - headache
 - dizziness
 - sore throat
 - runny nose

IMPORTANT SAFETY INFORMATION FOR AIRDUO DIGIHALER AND ARMONAIR DIGIHALER

- Common side effects of AIRDUO DIGIHALER and ARMONAIR DIGIHALER include:**
 - thrush in your mouth or throat. Rinse your mouth with water without swallowing after use to help prevent this.
 - Headache
 - cough
- Additionally, AIRDUO DIGIHALER has common side effects of:**
 - back pain
 - infection of nose and throat (nasopharyngitis)

Other Notifications and Digihalter App Capabilities

- The Digihalter app has the following in-app capabilities:
 - **Create Report:** Create your own Digihalter report and bring it to your next healthcare provider visit
 - **Self-Assessment:** Use the Daily Self-Assessment tool to help track how you feel over time
 - **Digihalers:** Make sure you bring your inhaler(s) within range of your smartphone regularly to capture your inhaler events in the app
 - **Environment:** Access real-time environment and weather information available for your location
- **System Messages:** These messages remind you of things you may need to do, including checking your dose counter and Internet connection
- **Banners:** These messages may appear in a blue bar at the top of the Home Screen, including
 - Turn on Bluetooth to Connect Digihalers
 - Your Phone's Clock Is Not Synced
 - Turn on Location Services
- **Inhaler Connectivity:** These messages will appear to inform you about your inhaler's connection to your app

IMPORTANT SAFETY INFORMATION FOR PROAIR DIGIHALER

- These are not all of the possible side effects of **PROAIR DIGIHALER**. For more information, ask your healthcare provider or pharmacist

IMPORTANT SAFETY INFORMATION FOR AIRDUO DIGIHALER AND ARMONAIR DIGIHALER

- **ARMONAIR DIGIHALER** has common side effects of:
 - upper respiratory tract infection
 - infection or inflammation of nose and throat (nasopharyngitis)
- These are not all the possible side effects of **AIRDUO DIGIHALER** and **ARMONAIR DIGIHALER**. Call your healthcare provider for medical advice about side effects.

How to Share Digihaler Reports

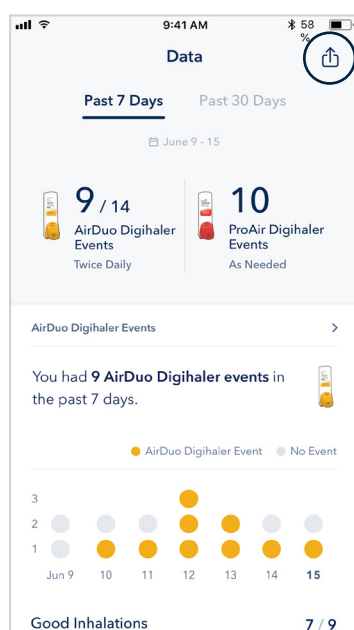


Figure L

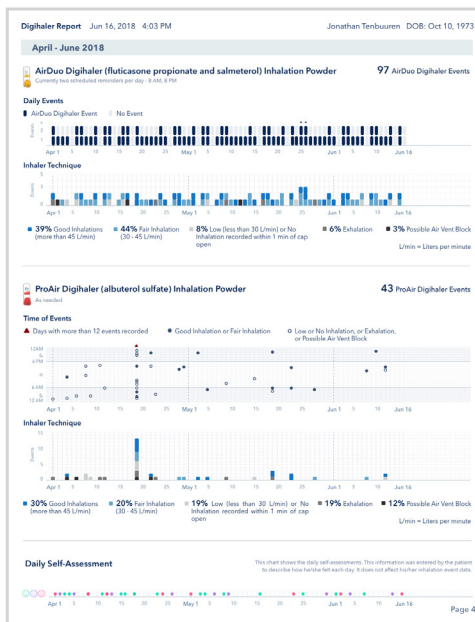


Figure M

- Tap on the “Share” icon in the top right of the Data Screen to create a PDF report (see Figure L)
- You will be given the option to add your name and/or date of birth to the Report Cover page. If you choose to skip this step, the report will not include this information
- Select the way you want to share your report. Your options are via text, email, or AirDrop®
- The report will consist of 3 main sections:
 - **Cover page:** This page defines the different types of inhaler events and how inhalation airflow is measured. You can also see the Approved Uses, initial Important Safety Information, and link to Prescribing Information for all scanned inhalers
 - **Important Safety Information pages:** These pages provide Important Safety Information and links to Prescribing Information for all scanned inhalers
 - **Report data page(s):** These pages contain the actual reports of your inhaler events (see Figure M). Each page displays 3 months of data for your healthcare provider or caregiver to view
- Remember, these reports are not intended for self-diagnosing purposes, but you can review them with your healthcare provider, if desired. Make sure you bring your inhaler within range of your smartphone to see the full details of your inhaler events each day
- There is no evidence the use of the app leads to improved clinical outcomes, including safety and effectiveness

IMPORTANT SAFETY INFORMATION FOR PROAIR DIGIHALER

- You are encouraged to report negative side effects of prescription drugs to the FDA. Visit www.fda.gov/medwatch, or call 1-800-FDA-1088

IMPORTANT SAFETY INFORMATION FOR AIRDUO DIGIHALER AND ARMONAIR DIGIHALER

- You are encouraged to report negative side effects of prescription drugs to the FDA. Visit www.fda.gov/medwatch, or call 1-800-FDA-1088.

Important Notes About the Inhalers in the Digihalter Family

If you are having trouble setting up your Digihalter app, go to www.digihalter.com, or call Teva at 1-888-603-0788 for assistance.

It is possible that your inhalation notification may not show up immediately due to wireless transmission interruptions or other issues.

You may need to open the app to see notifications.

Do not bring your inhaler(s) into an MRI or CT/X-ray scan room.

Do not expose your inhaler to other known sources of electromagnetic interference (such as diathermy and RF ID systems). Your inhaler can be taken through standard security systems, such as metal detectors.

The electronic module complies with part 15 of the FCC Rules. Operation is subject to the following 2 conditions:

- (1) The electronic module may not cause harmful interference, and
- (2) The electronic module must accept any interference received, including interference that may cause undesired operation.

Changes or modifications to the electronic module not expressly approved by Teva could void the user's authority to operate the equipment.

IP22 (protected against solid objects over 12.5 mm and direct sprays of water up to 15° from the vertical).

The Digihalter Bluetooth signal is discoverable, so inhaler ID (for example, Teva Inhaler) may be viewable by other mobile devices nearby. However, your inhaler use data is only sent to mobile devices that have been authenticated through the QR code.

SAFETY:

- IEC 60601-1:2005 + AMD 1:2012
- IEC 60601-1-11:2015

ELECTROMAGNETIC COMPATIBILITY:

- IEC 60601-1-2:2014
- ETSI EN 301 489-1 V2.2.1
- ETSI EN 301 489-1 V1.9.2
- 47 CFR Part 15 FCC 15 Class B
- ANSI C63.27™/D2.1
- CISPR 11 Group 1 Class B
- CISPR 22 - Class B, 1-6 GHz

Immunity Test	IEC 60601 Test Level	Compliance Level
Electrostatic Discharge (ESD) IEC 61000-4-2:2009	± 8kV Contact ± 15 kV Air	± 8kV Contact ± 15 kV Air
Radiated, radio-frequency, electromagnetic field immunity test IEC 61000-4-3:2006 + A1:2008+A2:2010	80-2700MHz :10 V/m, 80% AM, 1 kHz 385MHz :27 V/m, PM, 18Hz 450MHz :28 V/m, FM+/-5kHz dev, 1 kHz 710, 745, 780MHz :9 V/m, PM, 217 Hz 810, 870, 930MHz :28 V/m, PM, 18Hz 1720, 1845, 1970MHz :28 V/m, PM, 217 Hz 2450MHz :28 V/m, PM, 217 Hz 5240, 5500, 5785MHz :9 V/m, PM, 217 Hz	80-2700MHz :10 V/m, 80% AM, 1 kHz 385MHz :27 V/m, PM, 18Hz 450MHz :28 V/m, FM+/-5kHz dev, 1 kHz 710, 745, 780MHz :9 V/m, PM, 217 Hz 810, 870, 930MHz :28 V/m, PM, 18Hz 1720, 1845, 1970MHz :28 V/m, PM, 217 Hz 2450MHz :28 V/m, PM, 217 Hz 5240, 5500, 5785MHz :9 V/m, PM, 217 Hz

**ProAir®
digihalter®**
(albuterol sulfate 117 mcg)
Inhalation Powder

**AirDuo®
digihalter®**
(fluticasone propionate
113 mcg and salmeterol
14 mcg) Inhalation Powder

**ArmonAir®
digihalter®**
(fluticasone propionate
113 mcg) Inhalation Powder

The Bluetooth® word mark and logos are registered trademarks owned by the Bluetooth SIG, Inc. and any use of such marks by Teva is under license. Other trademarks and trade names are those of their respective owners.

Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries and regions. App Store is a service mark of Apple Inc.

Google Play and the Google Play logo are trademarks of Google LLC.

AirDrop and iPhone are registered trademarks of Apple Inc.

©2020 Teva Respiratory, LLC; All rights reserved.



ADH-40122
08/2020