





Welcome to Digihaler technology

Learn about ProAir Digihaler, AirDuo Digihaler, ArmonAir Digihaler, and the Digihaler app.









Get to know the Digihaler app and family of inhalers

You were recently prescribed one or more inhalers in the Digihaler family, which means you can now use Digihaler technology to help inform treatment conversations with your doctor.



Connection to the app is required for transmission of data but is not required for delivery of the medicine from the inhaler. There is no evidence the use of Digihaler technology leads to improved clinical outcomes, including safety and effectiveness.

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER®

 Do not use PROAIR DIGIHALER if you are allergic to albuterol sulfate, lactose, milk proteins, or any of the ingredients in PROAIR DIGIHALER. Ask your healthcare provider if you have any questions or are not sure

Learn about the medications

Whether you've been prescribed a rescue inhaler, a maintenance inhaler, or both, it's important to know that each have different uses.



Rescue medication

- ProAir Digihaler is a prescription medicine used in people 4 years of age and older to:
 - treat or prevent bronchospasm in people who have reversible obstructive airway disease
 - prevent exercise-induced bronchospasm



Maintenance medications

- AirDuo Digihaler is a prescription medicine used to control symptoms of asthma and to prevent symptoms such as wheezing in people 12 years of age and older
- ArmonAir Digihaler is a prescription medicine for the long-term treatment of asthma in patients 12 years and older
- AirDuo Digihaler and ArmonAir Digihaler are not used to relieve sudden breathing problems from asthma and won't replace a rescue inhaler

IMPORTANT SAFETY INFORMATION FOR AIRDUO® DIGIHALER® AND ARMONAIR® DIGIHALER®

AIRDUO DIGIHALER contains salmeterol. Long-acting beta₂-agonist (LABA) medicines such as salmeterol when used alone increase the risk of hospitalizations and death from asthma problems. AIRDUO DIGIHALER contains an inhaled corticosteroid (ICS) and a LABA. When an ICS and a LABA are used together, there is not a significant increased risk in hospitalizations and death from asthma problems.

Please see additional Important Safety Information throughout and full Prescribing Information for <u>ProAir Digihaler</u>, <u>AirDuo Digihaler</u>, and <u>ArmonAir Digihaler</u>.







(fluticasone propionate 113 mcg and salmeterol 14 mcg) Inhalation Powder

Team up with Digihaler technology

In addition to delivering medication, your inhaler has built-in sensors that provide objective data on inhaler events. It also has innovative technology that may help you understand how you use your inhaler. Inhaler use is recorded as an event when you open the cap or inhale.

The built-in sensors inside your inhaler can:

- Capture when and how often you use your inhaler
- Measure your inhalation airflow, or what your doctor may call inspiratory flow, which means how strongly you breathe in



You can view this data in the Digihaler app and share it with your doctor if you choose, which may help you have more informed treatment conversations.


IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® CONTINUED

- Before using PROAIR DIGIHALER, tell your healthcare provider about all of your medical conditions, including if you:
 - have heart problems
 - have high blood pressure (hypertension)
 - have convulsions (seizures)
 - have thyroid problems
 - have diabetes
 - have low potassium levels in your blood
 - are pregnant or planning to become pregnant
 - are breastfeeding or planning to breastfeed

Digihaler technology puts information in the palm of your hand.



Data is stored inside your inhaler, then sent to the Digihaler app using Bluetooth® wireless technology where it is displayed as an "event." It is referred to as an event in the Digihaler app because it is a record of what happened when you used your inhaler.

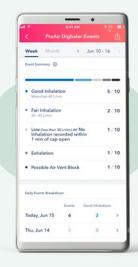

The app categorizes your events as:

- Good inhalation, measured as more than 45 liters per minute
- Fair inhalation, measured as 30-45 liters per minute
- Low or no inhalation. measured as less than 30 liters per minute
- Exhalation, or possible air vent block

IMPORTANT SAFETY INFORMATION FOR AIRDUO® DIGIHALER® AND ARMONAIR® DIGIHALER® CONTINUED

- Do not use AIRDUO DIGIHALER or ARMONAIR DIGIHALER to treat sudden breathing problems from asthma. Always have a rescue inhaler with you to treat sudden symptoms.
- Do not use AIRDUO DIGIHALER or ARMONAIR DIGIHALER if you have a severe allergy to milk proteins or if you are allergic to any of the ingredients in the product. Ask your healthcare provider if you are not sure.
- Do not use AIRDUO DIGIHALER or ARMONAIR DIGIHALER more often than prescribed.

Please see additional Important Safety Information throughout and full Prescribing Information for ProAir Digihaler, AirDuo Digihaler, and ArmonAir Digihaler.







Get started in a few steps



Download

- Open the iTunes App Store® or Google Play on your mobile device
- Search for **Digihaler**
- Tap the **Get** button to download the Digihaler app



2 Register

 Open the app, and follow the on-screen instructions to set up your account



Scan

- Find the black QR code on the top of your inhaler
- In the app, tap the
 Scan button
- Point the camera of your phone at the QR code

When your Digihaler is registered with the app, your screen will say Your Digihaler is registered



Watch a video to learn how to set up your inhaler and app at digihaler.com/support/patient-resources

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® CONTINUED

- Tell your healthcare provider about all the medicines you take, especially:
 - other inhaled medicines or asthma medicines
 - beta blocker medicines.
 - diuretics
 - digoxin
 - monoamine oxidase inhibitors
 - tricyclic antidepressants

IMPORTANT SAFETY INFORMATION FOR AIRDUO® DIGIHALER® AND ARMONAIR® DIGIHALER® CONTINUED.


- Do not take AIRDUO DIGIHALER with other medicines that contain a LABA for any reason.
- Tell your healthcare provider about all the medicines you take and about all of your health conditions.
- Do not stop using ARMONAIR DIGIHALER, even if you are feeling better, unless your healthcare provider tells you to. If you miss a dose of ARMONAIR DIGIHALER, just skip that dose. Take your next dose at your usual time. Do not take 2 doses at 1 time.

Please see additional Important Safety Information throughout and full Prescribing Information for ProAir Digihaler, AirDuo Digihaler, and ArmonAir Digihaler.







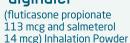



The Digihaler app features data screens and notifications to help you stay informed.

In the Digihaler app, you can view:

- The date and time of events for rescue and/or maintenance inhalers
- Your inhalation airflow, which means how strongly you breathe in


The app also allows you to set up notifications that can tell you about your rescue inhaler events, or remind you to take your maintenance medication.

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® CONTINUED

- Do not increase your dose or take extra doses of PROAIR DIGIHALER without first talking to your healthcare provider
- Get medical help right away if **PROAIR DIGIHALER** no longer helps your symptoms, your symptoms get worse or you need to use your inhaler more often

See how often you're using your rescue inhaler

The app will notify you if you have more than 12 ProAir Digihaler events in one day. Inhaler use is recorded as an event when you open the cap or inhale.

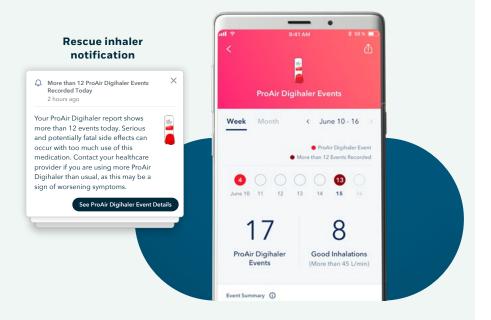

Reviewing your ProAir Digihaler events may help you:

- Understand if there are any trends in your rescue inhaler events
- Inform your doctor if there is an increase in your inhaler events, if you choose to share your information

IMPORTANT SAFETY INFORMATION FOR AIRDUO® DIGIHALER® AND ARMONAIR® DIGIHALER® CONTINUED

- AIRDUO DIGIHALER and ARMONAIR DIGIHALER can cause serious side effects, including:
 - Fungal infection in your mouth or throat (thrush). Rinse your mouth with water without swallowing after using AIRDUO DIGIHALER or ARMONAIR **DIGIHALER** to help reduce your chance of getting thrush. (serious side effects continued on page 11)

Please see additional Important Safety Information throughout and full Prescribing Information for ProAir Digihaler, AirDuo Digihaler, and ArmonAir Digihaler.








(fluticasone propionate 113 mcg and salmeterol 14 mcg) Inhalation Powder

Keep track of your maintenance inhaler events

You'll also receive reminders to use your maintenance inhaler, if you are prescribed one.

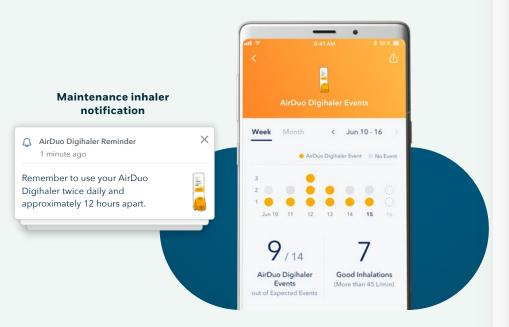

Refer back to weekly and monthly views in the app to see if you took your medication as prescribed. Inhaler use is recorded as an event when you open the cap or inhale.


IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® CONTINUED

 While you are using PROAIR DIGIHALER, do not use other inhaled rescue medicines and asthma medicines unless your healthcare provider tells you to do so

Review your inhalation airflow

If 6 of your last 12 inhaler events are recorded as "fair,"
"low or no inhalation," "exhalation," or "possibile air vent block,"
you will receive a notification that reminds you to review the
instructions on how to use your inhaler correctly.

Note that if you open the cap but do not inhale, your inhaler will record an event as "low or no inhalation."

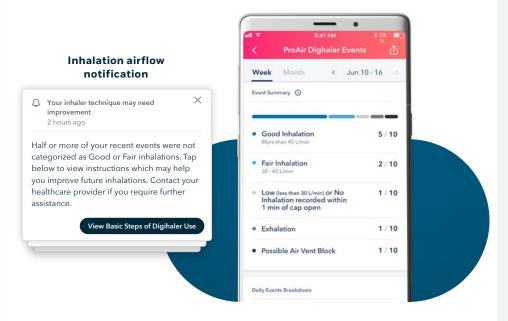

The weekly and monthly data views can help you learn more about your inhaler events over time. Viewing your events with your doctor can help you see if your inhaler technique might need to be assessed.

IMPORTANT SAFETY INFORMATION FOR AIRDUO® DIGIHALER® AND ARMONAIR® DIGIHALER® CONTINUED

(serious side effects continued)

 Weakened immune system and increased chance of getting infections (immunosuppression). You should avoid exposure to chickenpox and measles, and, if exposed, tell your healthcare provider right away.
 Worsening of existing tuberculosis, fungal, bacterial, viral, or parasitic infections, or herpes infection of the eye (ocular herpes simplex) may occur. (serious side effects continued on page 13)

Please see additional Important Safety Information throughout and full Prescribing Information for <u>ProAir Digihaler</u>, <u>AirDuo Digihaler</u>, and <u>ArmonAir Digihaler</u>.







Make Digihaler data part of the conversation

If you choose to share your inhaler event information with your doctor, it may help both of you have more informed treatment conversations.

You can share information with your doctor in three ways:



PDF

The Digihaler app can generate a PDF of your report that you can share with your doctor through text, email, or AirDrop.



Printed Report

Print your PDF report and bring it to your next appointment.



Data Screens

Share your data screens directly in the app by showing your phone to your doctor.

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® CONTINUED

- PROAIR DIGIHALER may cause serious side effects, including:
 - worsening trouble breathing, coughing and wheezing (paradoxical bronchospasm). If this happens, stop using PROAIR DIGIHALER and call your healthcare provider or get emergency help right away. This is more likely to happen with your first use of a new asthma inhalation medicine
 - heart problems, including faster heart rate and higher blood pressure
 - possible death in people with asthma who use too much PROAIR DIGIHALER
 - allergic reactions. Call your healthcare provider right away if you have the following symptoms of an allergic reaction:
 - > itchy skin
 - > swelling beneath your skin or in your throat
 - > rash
 - > worsening trouble breathing
 - changes in laboratory blood values (sugar, potassium)

Prepare for your next appointment

Have an appointment coming up? Here are some suggestions to help you have an informed discussion with your doctor:

- Review the information in your app before your appointment
- If you'd like to share your information with your doctor, you can show your app screens in-person, or export and print a PDF report to bring to your appointment
- Ask yourself a few questions before a consultation with your doctor. For example:
 - Have I been using my rescue inhaler more frequently?
 - Have I been using my maintenance inhaler regularly?
 - Are there changes in my inhaler event patterns, such as time of day?
 - Have there been changes in my inhalation airflow over time?



Inhaler use is recorded as an event when you open the cap or inhale.

IMPORTANT SAFETY INFORMATION FOR AIRDUO® DIGIHALER® AND ARMONAIR® DIGIHALER® CONTINUED

(serious side effects continued)

- Reduced adrenal function. This can happen when you stop taking an oral corticosteroid (such as prednisone) and start taking a medicine containing an ICS (such as AIRDUO DIGIHALER or ARMONAIR DIGIHALER). During this transition period, when your body is under stress such as from fever, trauma (such as a car accident), infection, or surgery, adrenal insufficiency can get worse and may cause death. Symptoms of adrenal insufficiency include:
 - > feeling tired
 - > lack of energy
 - > Weakness
 - > nausea and vomiting
 - > low blood pressure

Please see additional Important Safety Information throughout and full Prescribing Information for ProAir Digihaler, AirDuo Digihaler, and ArmonAir Digihaler.







(fluticasone propionate 113 mcg and salmeterol 14 mcg) Inhalation Powder

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® CONTINUED

- The most common side effects of PROAIR DIGIHALER include:
 - back pain
 - body aches and pain
 - upset stomach
 - sinus headache
 - urinary tract infection
 - your heart feels like it is pounding or racing (palpitations)
 - chest pain
 - fast heart rate
 - shakiness
 - nervousness
 - headache
 - dizziness
 - sore throat
 - runny nose
- These are not all of the possible side effects of **PROAIR DIGIHALER.** For more information, ask your healthcare provider or pharmacist
- You are encouraged to report negative side effects of prescription drugs to the FDA. Visit www.fda.gov/medwatch, or call 1-800-FDA-1088

Please see additional Important Safety Information throughout and full Prescribing Information for **PROAIR DIGIHALER**.

IMPORTANT SAFETY INFORMATION FOR AIRDUO® DIGIHALER® AND ARMONAIR® DIGIHALER® CONTINUED

- For AIRDUO DIGIHALER, there may be sudden breathing problems immediately after inhaling your medicine. If you have sudden breathing problems immediately after inhaling your medicine, stop using AIRDUO DIGIHALER and call your healthcare provider right away.
- Serious allergic reactions. Stop using AIRDUO DIGIHALER or ARMONAIR DIGIHALER and call your healthcare provider or get emergency medical care if you get any of the following symptoms of a serious allergic reaction:
 - > rash
 - > hives
 - > swelling of your face, mouth, and tongue
 - > breathing problems

- AIRDUO DIGIHALER can also cause additional serious side effects, including:
 - Effects on the heart
 - > increased blood pressure
 - > a fast or irregular heartbeat
 - > chest pain
 - Effects on the nervous system
 - > Tremor
 - > nervousness
- AIRDUO DIGIHALER and ARMONAIR DIGIHALER can cause:
 - Bone thinning or weakness (osteoporosis)
 - **Slowed growth in children.** A child's growth should be checked often.
 - Eye problems including glaucoma and cataracts. You should have regular eye exams while using AIRDUO DIGIHALER or ARMONAIR DIGIHALER.
- For AIRDUO DIGIHALER, changes may occur in laboratory blood values (sugar, potassium, certain types of white blood cells)
- For ARMONAIR DIGIHALER increased wheezing (bronchospasm) may occur. Increased wheezing can happen right away after using ARMONAIR DIGIHALER. If this occurs, stop using ARMONAIR DIGIHALER and call your healthcare provider. Always have a rescue inhaler with you to treat sudden wheezing.
- Common side effects of AIRDUO DIGIHALER and ARMONAIR DIGIHALER include:
 - thrush in your mouth or throat. Rinse your mouth with water without swallowing after use to help prevent this.
 - Headache
 - cough
- Additionally, AIRDUO DIGIHALER has common side effects of:
 - back pain
 - infection of nose and throat (nasopharyngitis)

ARMONAIR DIGIHALER has common side effects of:

- upper respiratory tract infection
- infection or inflammation of nose and throat (nasopharyngitis)
- These are not all the possible side effects of AIRDUO DIGIHALER and ARMONAIR DIGIHALER. Call your healthcare provider for medical advice about side effects.
- You are encouraged to report negative side effects of prescription drugs to the FDA. Visit www.fda.gov/medwatch. or call 1-800-FDA-1088.

Please see additional Important Safety Information throughout and full Prescribing Information for <u>AIRDUO DIGIHALER</u> and <u>ARMONAIR DIGIHALER</u>.







Digihaler support every step of the way



If you have questions about your inhaler or the Digihaler app, visit www.digihaler.com/support



Customer support is available by calling **+1(888) 603-0788** Monday through Friday

Digihaler Savings Program

You may be able to save on your prescription(s). The program is available for the entire Digihaler family of inhalers.

Visit <u>digihaler.com</u> to see full terms and conditions



Not a real card

Please note, this offer is not available for patients eligible for Medicare, Medicaid, or any other form of government insurance.

Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries and regions. App Store is a service mark of Apple Inc. AirDrop and iPhone are registered trademarks of Apple Inc. The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Teva is under license. Other trademarks and trade names are those of their respective owners. Google Play and the Google Play logo are trademarks of Google LLC.

